제품 보증서 QUALITY GUARANTEE

PRODUCT (제품)	
Model No. (모델명)	
Serial No. (일련번호)	
Purchase date (구입일자)	
CUSTOMER (고객)	
Company (회사명)	
Address (주소)	
Tel. (전화)	
E-Mail (이메일)	

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English

- The legal warranty period for the perfect functioning of purchased goods is 12 months from date of purchase.
- For the manufacture of this product, only first class, proved material was used with the utmost care. Anyway, should you recognize a failure, based on material errors or incorrect assembly can be attributed. We will remove the source of this error immediately and free of charge. You have to enforce your claims in written from during this warranty period.
- Defects due to improper handling, installation error, excessive usage, misapplication or damage resulting from inexpert repairs or modifications by either your-self or by any third party are not covered by the warranty.
- Also defects, which are based on normal abrasion are excluded from warranty.
- In particular, we do not assume, as far as permitted by law, no liability for consequential damages who are caused by the device.
- Claims of other species, such as rescission, mitigation or compensation are excluded by the warranty.
- The warranty is only valid to the company of the first purchaser.
- Be sure to keep your sales receipt and the completed warranty slip in a safe place ; this will facilitate any future warranty claims.
- Shipment of defect devices, has to occur exempt free from postage for us.

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1. IMPORTANT INFORMATION

Read this instruction before starting up the machine carefully to avoid incorrect operation or other damage. If other language version is required, please request our agent in your area. Duplication and distribution of this instructions require MICRO-NX's prior consent.

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Manufacture has right to do modification and improvement for the product's quality.

But It does not imply any right to retrofitting existing units.

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MICRO-NX is not responsible for damage caused by below things.

- Damage caused by nonfulfillment of this instruction.
- External cause like improper installation etc
- Improper use,
- Repair performed by unqualified person.

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
MICRO-NX recommend using only original spare parts supplied by us for operation and repair.

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
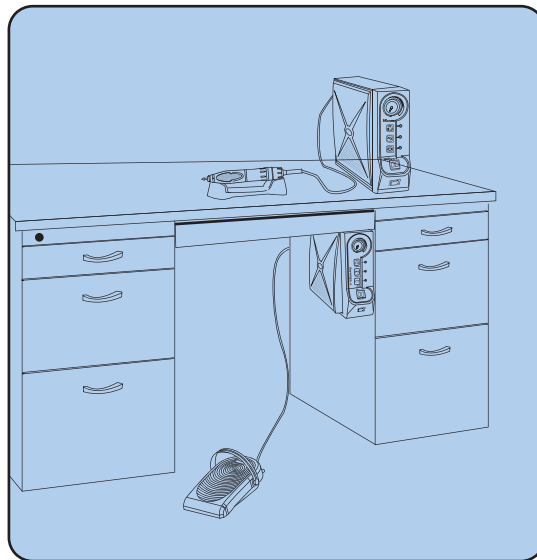
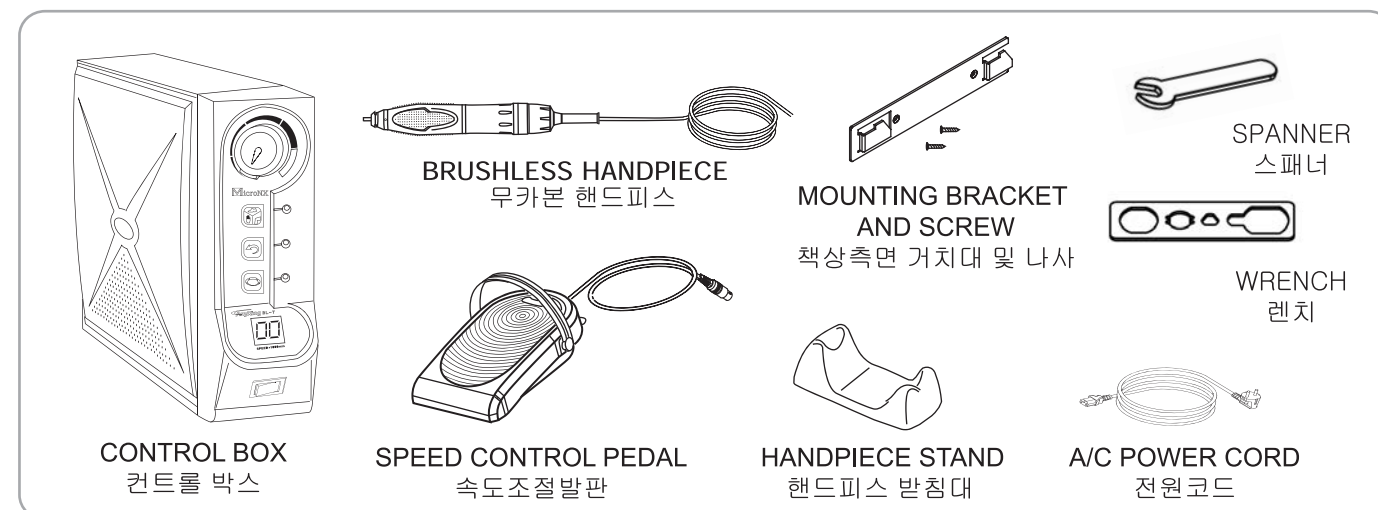
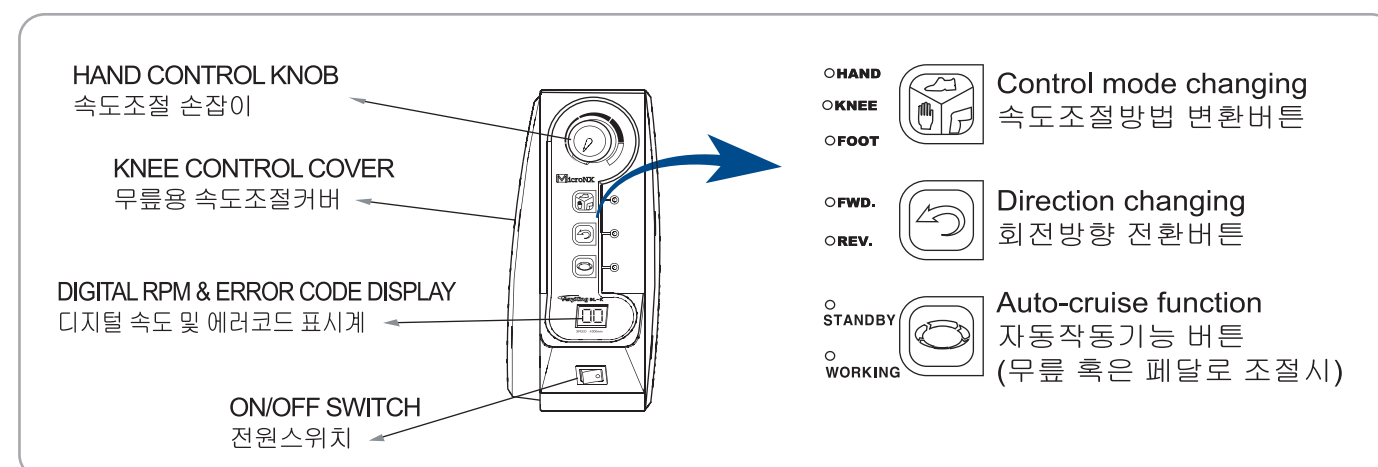
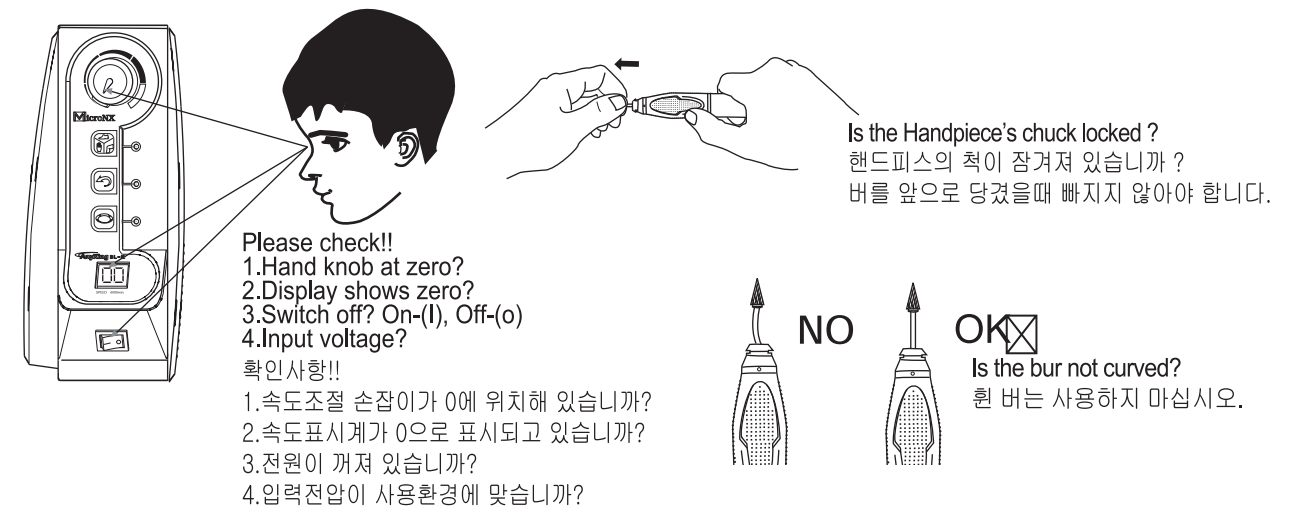
Check points before operation / 작동 전 확인사항

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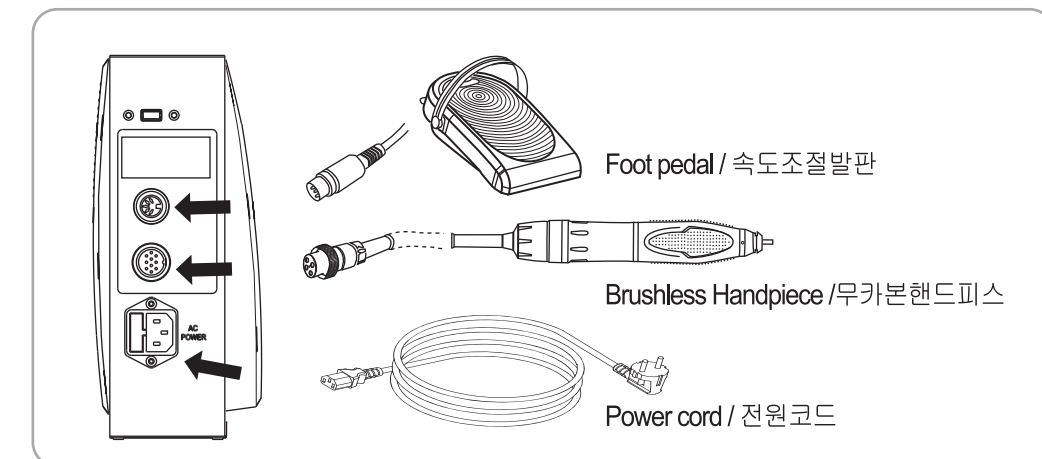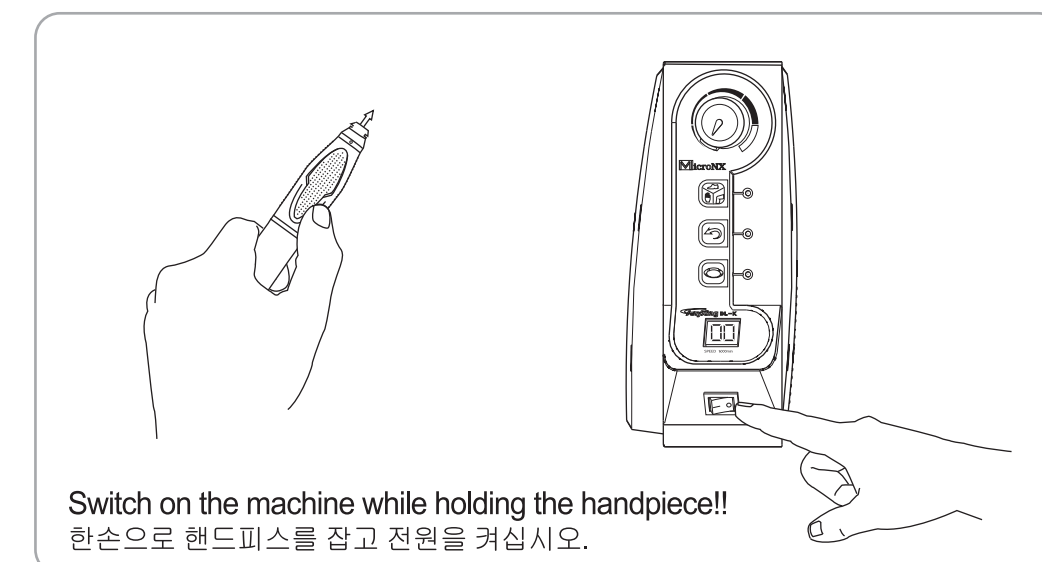STEP 2 Switch on / 전원켜기

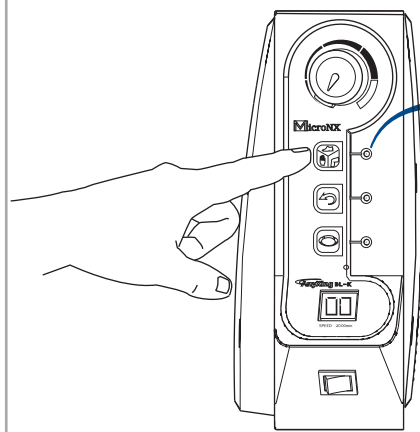
STEP 3 Selecting speed control mode / 속도조절방법 선택

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Press first button  to change speed control mode.

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Red light-Hand control mode
Yellow light-Knee control mode
Green light-Foot control mode

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Maximum Speed setting function !!

- When you control the speed by knee or foot pedal, the speed will increase up to a speed which hand-knob points out.

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Auto-Cruise function !!

- When you control the speed by knee or foot pedal, press third button  and wait for 3~4 seconds !! Then the machine will work automatically after beep sound.

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

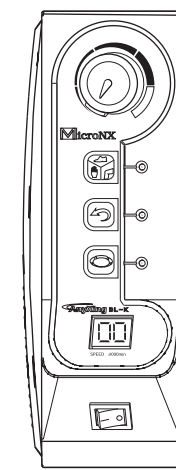
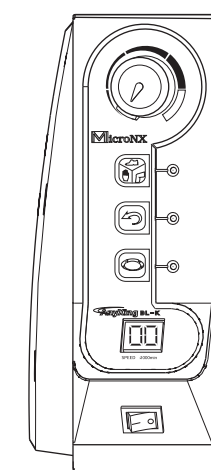
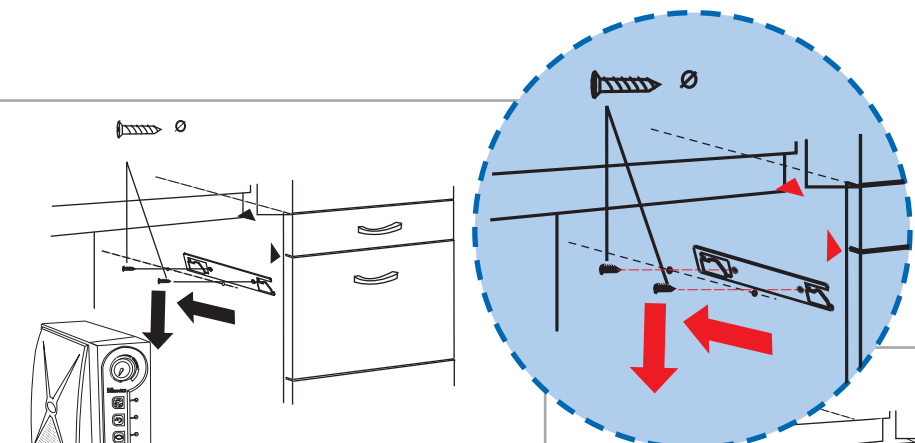
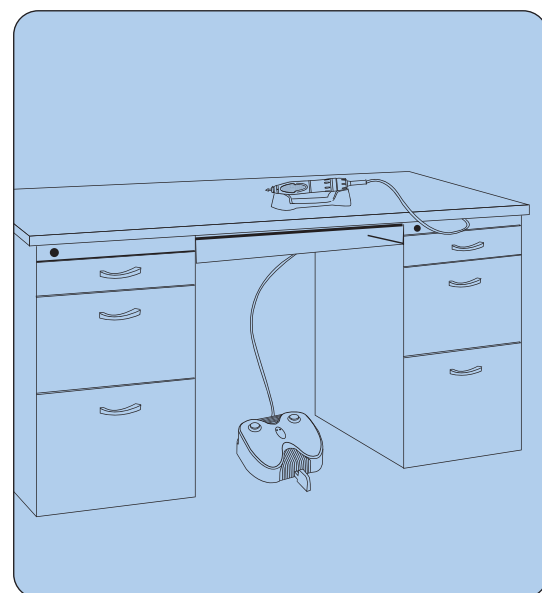
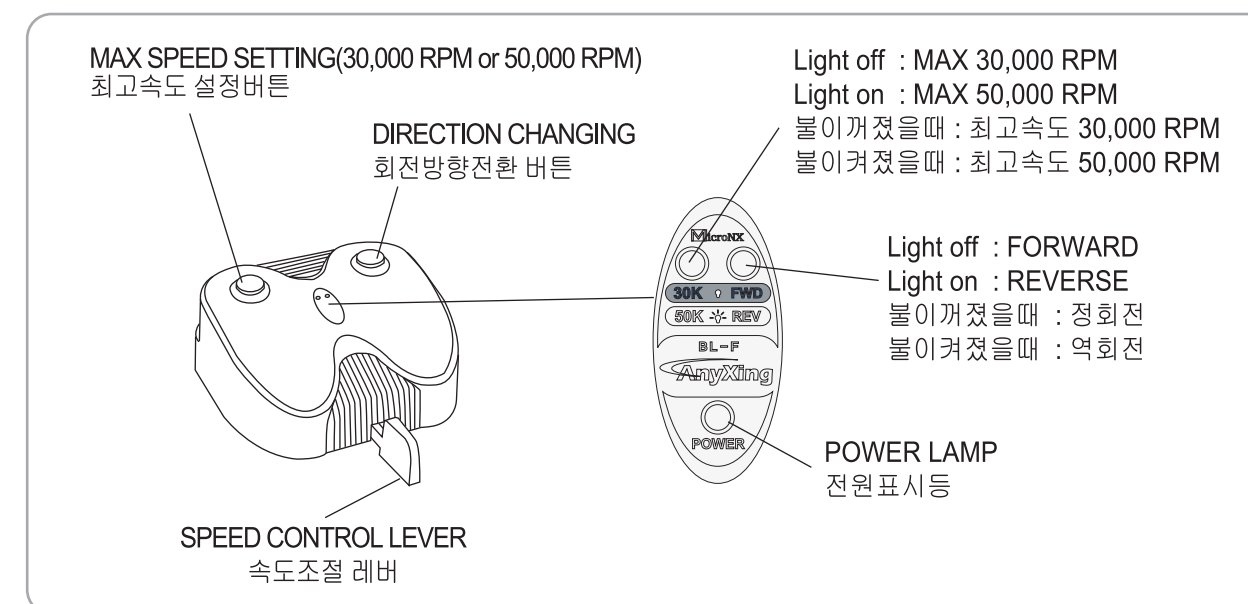
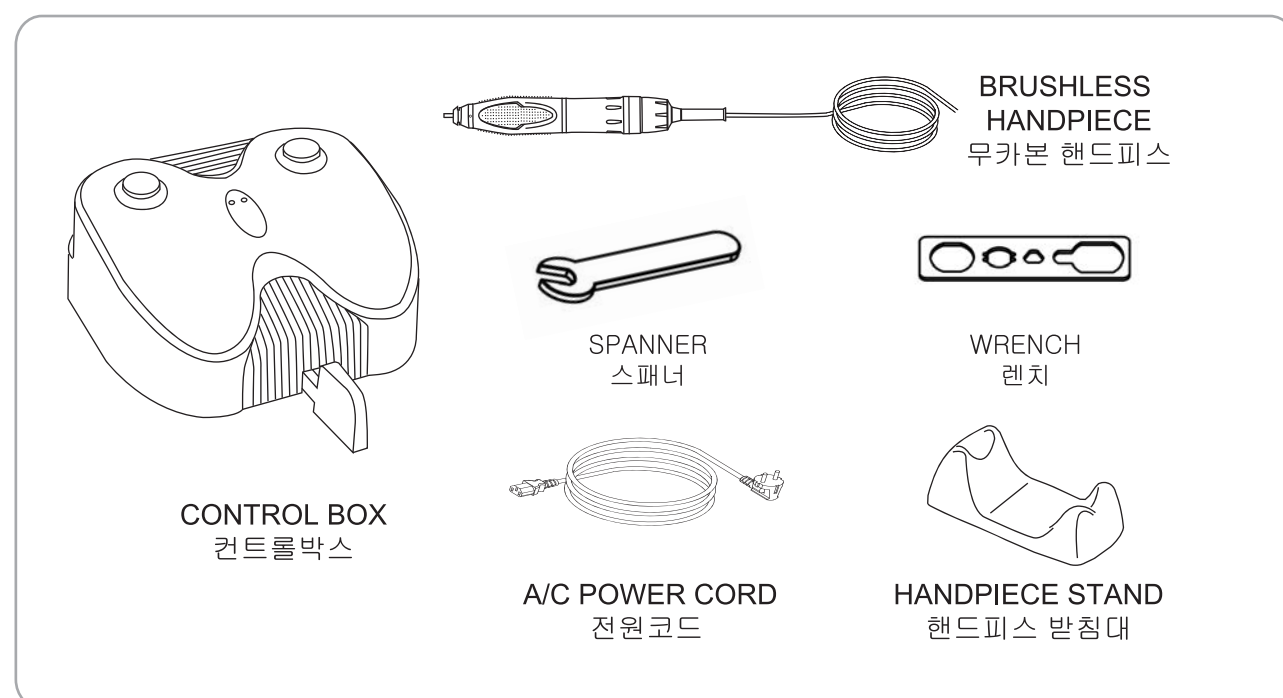
3-2 How to operate / 작동방법

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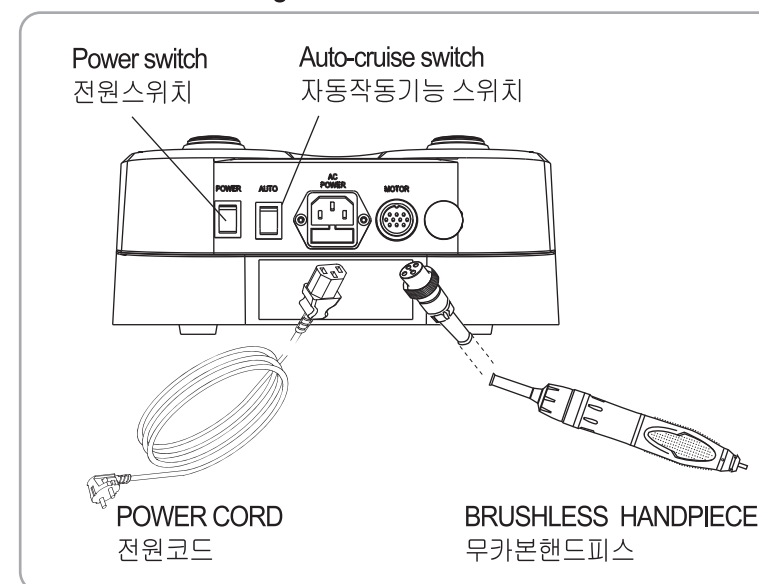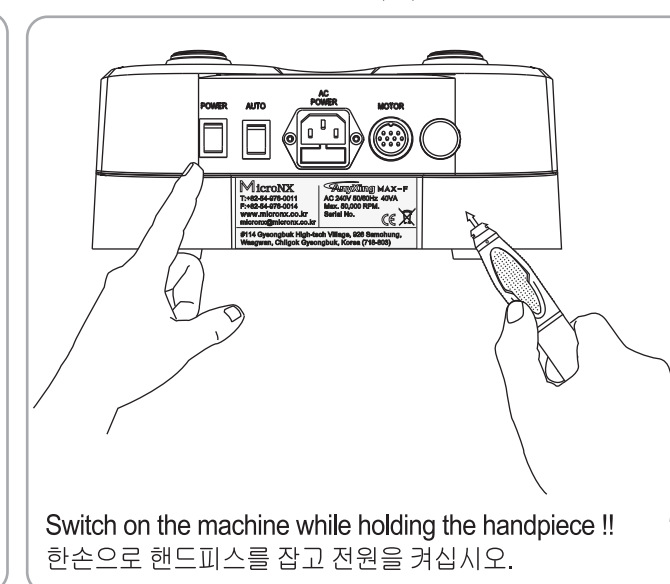4. RECOMMENDATION for SAFE and CONSTANT USE

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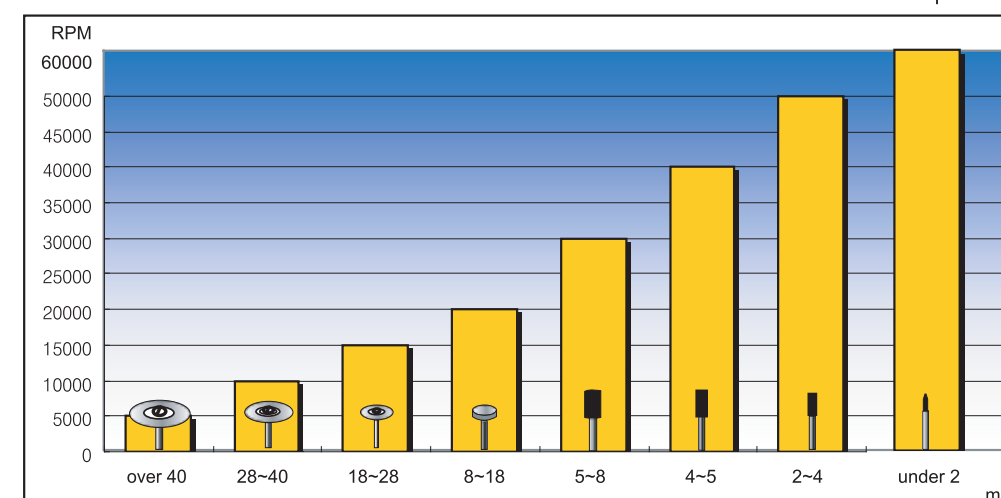
<Example Chart>


Notice! Definitely keep the permitted speed to prevent you and property from danger.

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
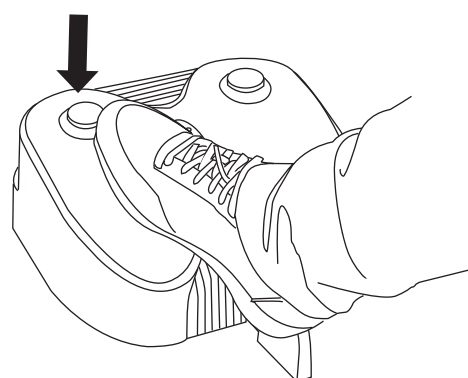
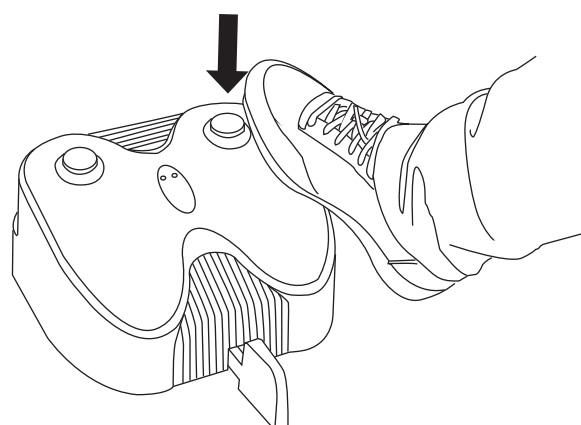
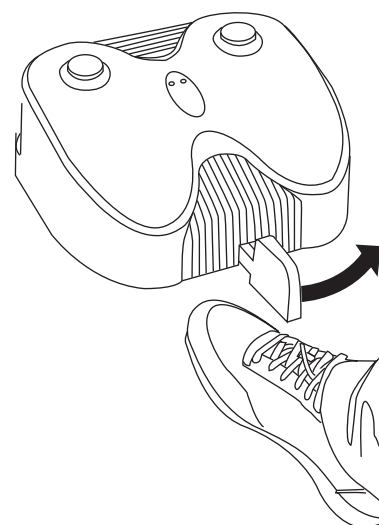
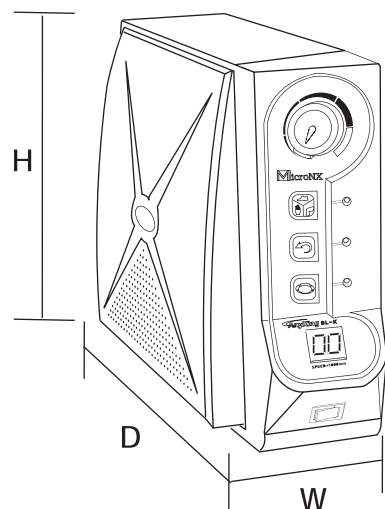
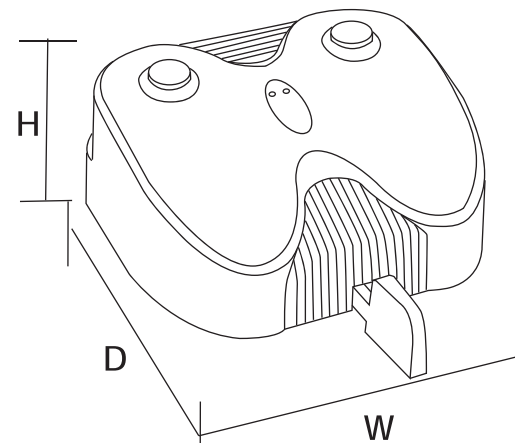
6. USE AND MAINTENANCE of HANDPIECE

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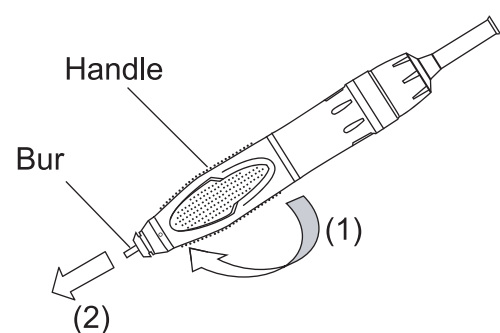6-1 Changing a burr / 버 교체방법

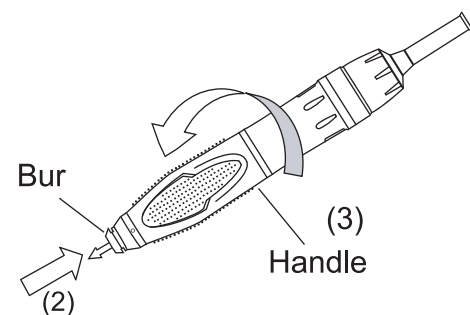(1) Turn left the handle to unlock the chuck.
(2) Remove an old burr and insert a new burr.
(3) Turn right the handle to lock the chuck.

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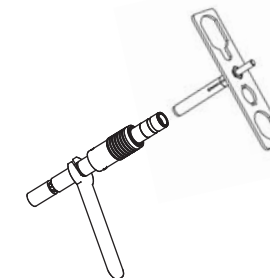
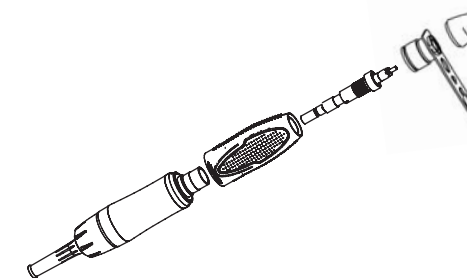
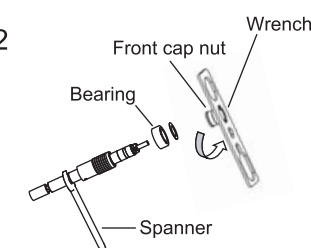
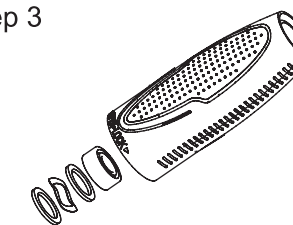
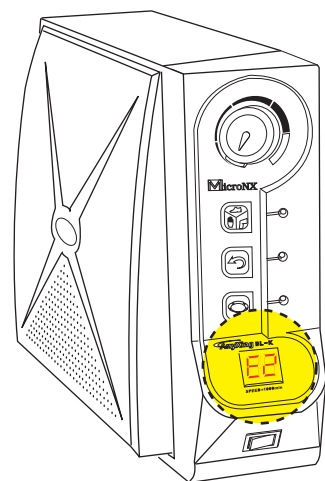
When there is problem on the machine, error code will be shown on the display.

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Problem	Check point	Solution
Power Lamp is not lighten when you turn on the power switch.	Is the power cord plug connected well?	Insert the power plug correctly.
	Isn't the fuse broken ?	Replace to new fuse.
	Trouble of power switch.	Please ask repair.
The motor does not work. (Please check the error code list.)	E1 It is fault of hall sensor in motor.	Please ask repair.
	E2 It is motor's fault.	1.Check electrical connection of motor 2.If this error code is displayed continuously, Please ask repair.
	E3 Have you used a handpiece for long time under high-load ?	Turn off the control unit, please keep it at cool place for 10 minutes and try again.
	E4 It is error on heat sink over 80℃	Same as above.
	E5 It is a transformer temperature error over 80℃	Please ask repair.
	E6 It is fault on circuit or transformer.	Please ask repair.
	E7 Is speed pedal's cord disconnected ?	1.Connect the pedal's cord correctly. 2.If this error code is displayed continuously, Please ask repair.
	E8 It is fault on a magnet inside knee control board.	Please ask repair.
Foot pedal does not work.	Is the cord of foot pedal connected well ?	Connect the pedal plug into a correct jack.
	Is Foot control mode selected ?	Select foot pedal control mode. It is green lamp on panel.
The handpiece makes noise.	Isn't foreign substance in the collect chuck?	Please remove it by asking repair to our agent in your area.
	Don't you use it at abnormal speed or Isn't the bur curved?	Replace the curved bur and use under permitted RPM for bur size.
	Check if the ball bearing is good condition.	Please ask repair.

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